INFINI®
PREMIUM FILLER

IT

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Descrizione

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EN INSTRUCTIONS FOR USE

Description

Infini Premium Filler is a sterile, injectable, apyrogenic, resorbable medical device made with cross-linked hyaluronic acid derived from bacterial fermentation. Infini Premium Filler is in the form of a homogeneous, transparent, colourless gel contained in a pre-filled, single-use, sterile syringe with Luer-lock fitting.

Composition

Infini Premium Filler S 20mg/ml: Concentrated cross-linked hyaluronic acid. Sterile saline solution: as needed.

Infini Premium Filler SOFT LIPS 20mg/ml: Concentrated cross-linked hyaluronic acid. Sterile saline solution: as needed.

Infini Premium Filler M 21mg/ml: Medium- concentration cross-linked hyaluronic acid. Sterile saline solution: as needed.

Infini Premium Filler L 24mg/ml: High-concentration cross-linked hyaluronic acid. Sterile saline solution: as needed.

Infini Premium Filler XL 26mg/ml: Extremely-concentrated cross-linked hyaluronic acid. Sterile saline solution: as needed.

Infini Premium Filler B BODY 20mg/ml: Concentrated cross-linked hyaluronic acid. Sterile saline solution: as needed.

Infini Premium Filler A AQUABOOSTER 20mg/ml: Low cross-linking concentrated hyaluronic acid. Sterile saline solution: as needed.

Action mechanism

The action of Infini Premium Filler consists in increasing the volume of the dermal-epidermal tissue based on the natural ability of the hydrophilic molecules of hyaluronic acid to bond with a much greater quantity of water than their weight. This fills the intradermal spaces and integrates the intercellular matrix, making the tissues firm. The cross-linking of the hyaluronic acid contained in the product makes the filling effect more stable and longer lasting.

Directions for use

Infini Premium Filler is intended for use as a temporary filler for the correction of skin defects such as wrinkles or scars. It is also recommended for pathological conditions such as lipoatrophy connected to infection from HIV and antiretroviral therapy.

In particular we recommend the following use

Infini Premium Filler S for the correction of superficial lines and wrinkles, from fine to medium. Particularly indicated for defects and areas: laugh lines, barcodes, periorcular wrinkles, scar sequelae (acne).

Infini Premium Filler SOFT LIPS for the correction of superficial lines and wrinkles, from fine to medium. Particularly indicated for defects and areas: laugh lines, barcodes, periorcular wrinkles, scar sequelae (acne)

Infini Premium Filler M for the correction of medium-superficial wrinkles on the face. Particularly indicated for defects and areas: nasolabial wrinkles, nose, lips, frown lines.

Infini Premium Filler L for the correction of deep facial wrinkles and folds. Particularly indicated for defects and areas: nasolabial folds, marionette lines, temples, remodelling facial contours, lips.

Infini Premium Filler XL for the correction of very deep wrinkles on the face and to restore facial volume. Particularly indicated for defects and areas: cheekbone, malar region, sub-zygomatic region, sub-malar region, chin.

Infini Premium Filler B BODY for the correction of wider-spread skin imperfections that affect large volumes and the body.

Infini Premium Filler A AQUABOOSTER for the correction of slight depressions and skin imperfections of the face. Particularly indicated for defects and areas: photoaging and chronoaging of the face, asphyctic and dehydrated skin.

Contraindications

Infini Premium Filler must not be used: for injections in the periorbital zone (eyelids, under eye bags); for injections in blood vessels; for augmentations of the mammary tissue or for bone, tendon, ligament or muscle implants. Infini Premium Filler must not be administered to patients with: previous

or current autoimmune diseases or being treated with immunotherapeutic drugs; proven tendency to form hypertrophic scars or keloids or pigmentation disorders; multiple serious allergies; infections and inflammatory processes in situ or near the area being treated; coagulation disorders; recognised hypersensitivity to hyaluronic acid. Infini Premium Filler must not be used by women who are pregnant, nursing or on people under the age of 18.

Warnings

Infini Premium Filler is only intended to be injected subcutaneously or intradermally and must not be injected intravascularly. In performing the injection procedure, there is a potential risk that the material gets inadvertently injected into the blood vessels. In rare cases this could cause vascular occlusion, abscess, necrosis or embolic phenomena. To ensure that the needle is not inside a blood vessel during normal use of the medical device, lightly aspirate with the syringe before proceeding with the injection. Infini Premium Filler was packaged to be used on one patient only and within the same session. Infini Premium Filler is supplied as a single-use sterile product and must not be re-sterilised.

Do not re-use.

Use Infini Premium Filler immediately after opening the blister pack and discard the syringe immediately after use, even if the entire contents have not been used. Do not use the product past the expiry date shown on the package or if said package is damaged, opened or stored improperly.

Do not use Infini Premium Filler in combination with other injectable products or mix it with other preparations before use.

Do not use Infini Premium Filler when the skin shows redness, swelling or changes in pigmentation; do not use Infini Premium Filler with inflammation, infection or fever in progress.

Patients taking medications to slow blood clotting may present with bruising or significant bleeding at the injection site. Patients taking anticoagulant medication should stop taking it, in agreement with their doctor, at least 14 days before getting Infini Premium Filler injections.

INCOMPATIBILITY:

There is a recognised incompatibility between hyaluronic acid and quaternary ammonium salts such as benzalkonium chloride. Therefore avoid contact of Infini Premium Filler with these substances or using medical-surgical material containing quaternary ammonium salts. Keep out of the reach of children.

Precautions for use

Only use Infini Premium Filler in compliance with these instructions for use. Infini Premium Filler is a colourless transparent gel: if the contents of the syringe appear altered or contain particles, do not use it and inform INFINI LAB SRL at the address below.

Do not use the device if the syringe or blister it is contained in is damaged. Follow all normal handling precautions associated with intradermal injection procedures. Injection is associated with an inherent risk of infection. The skin must be perfectly clean and disinfected before treatment.

Do not use on patients with ongoing infections or inflammatory processes near or at the injection site. It is recommended not to use Infini Premium Filler in areas where other implants have been performed and not to use it at the same time as any other injectable product. The patient should not wear makeup for the 24 hours following the injection and should be informed that, during the remission period, or until the swelling or redness has completely disappeared, he/she should avoid exposing the treated area to direct sunlight, UV lamps or cold. If a laser, chemical peel or dermabrasion treatment is carried out after using Infini Premium Filler, an inflammatory reaction may occur. For the same reason Infini Premium Filler should be used only when the sequelae of any previous treatment completely subsided.

After use, syringes and needles can pose a potential biological hazard. Handle with care and dispose of in accordance with approved medical practice.

Adverse reactions

Before starting treatment, the doctor must inform the patient that there are potential secondary effects related to intradermal implant that may occur immediately or after some time.

Following the use of Infini Premium Filler

Some injection-related adverse reactions may occur, which include pain, itching, altered pigmentation and tissue density in the area of the injection site and which normally resolve spontaneously within a week. Sometimes small oedemas may appear, which reabsorb in few days. Adverse reactions related to the use of the product, in consideration of the bibliographic studies of the existing literature and the scientific knowledge of the producer at the date of issue of this leaflet, can include the following: inflammatory reactions, itching, pain to the touch after injection lasting up to a week at most; bleeding or haematoma in rare cases; follicular papules-pustules that may form up to 4 weeks after treatment; allergic reactions to hyaluronic acid; hardening or appearance of nodules at the injection site; in very rare cases, granulomas or significant inflammatory reactions, wheals, necrosis, urticaria, which can appear even after 2-4 weeks.

IMPORTANT:

Patients should tell their doctor about any type of serious adverse reaction or otherwise not listed in this leaflet, and the doctor, in addition to prescribing the appropriate treatment for these reactions, has the obligation to the distributor INFINI LAB SRL and the manufacturer VIDA SRL.

How to use

Before starting treatment, the doctor must inform the patient about the composition, characteristics, precautions for use, warnings, contraindications and potential adverse reactions associated with this device. Furthermore, the doctor must first take a medical history and perform an evaluation of any previous immunological or allergic problems of the patient and ascertain compatibility with the use of Infini Premium Filler. Finally, the doctor must evaluate the skin areas involved in the treatment to ascertain patient suitability for the implant and the actual usefulness of such treatment.

Infini Premium Filler can only be administered by qualified medical personnel. For a correct final result of the treatment it is essential to proceed with a correct injection technique. Before the injection, disinfect the area to be treated with alcohol or other antiseptic solutions.

The product must be injected using a sterile syringe equipped with a fine needle.

**For Infini Premium Filler S, INFINI PREMIUM FILLER SOFT LIPS we recommend using a 30G needle,
for Infini Premium Filler M a 27G or 30G needle,
for Infini Premium Filler L a 27G or 25G needle,
for Infini Premium Filler XL a 25G needle,
for Infini Premium Filler B BODY an 18G cannula,
for Infini Premium Filler A AQUABOOSTER a 30G needle.**

However, the choice of needle and the method of injection is at the discretion of the doctor. The amount of product to be injected must be established according to the depth of the wrinkle or skin lesion and can be easily dosed thanks to the use of graduated syringes. Immediately after performing the injections, firm pressure and massage of the treated part should be applied/ carried out in order to promote homogeneous distribution of the product. Before each procedure the symmetry of the selected areas of the face must be carefully evaluated, planning to inject the same amount of material on the right and left, and at symmetrical points. Immediately before performing the implant, remove the syringe from the protective blister, unscrew the syringe cap and position the needle by screwing the needle cone onto the syringe Luer-lock attachment until you feel resistance.

For safe use of Infini Premium Filler it is important to use an appropriate needle and assemble the needle correctly on the syringe, as incorrect assembly may cause the needle to detach from the syringe during injection. Remove the protective cap from the needle only before injection.

Do not use needles that are not sterile, bent, warped or if packaging is not perfectly intact.

Before injection, expel air from the syringe until a drop of product comes out from the tip of the needle.

Insert the needle at an angle of 30 degrees, following exactly the course of the wrinkle or depression and inject the product into the middle dermis. If hyaluronic acid is administered too deeply, the final effect lasts less over time; on the contrary, if it is injected too superficially, nodules or granulomas can form.

Never apply excessive pressure on the syringe; inject Infini Premium Filler while the needle is slowly being retracted, depositing a homogeneous amount of product along the chosen site. Stop applying pressure to the syringe plunger before removing the needle from the skin to avoid spilling material from the injection point. In case of immediate swelling it is helpful to briefly apply cold compresses. Note: the graduation on the syringe is not provided for measuring purposes, but only as a visual check of the injected material.

Dosage

The dosage depends on the condition of the patient's skin and the depth of the area to be treated. Optimal results can be obtained by repeating the treatment on a regular basis over time, taking into account the different factors that contribute to the long-lasting efficiency of the injection, such as lifestyle, physical activity, exposure to sun rays or tanning lamps, age and appropriate filling technique.

Packaging

One package of Infini Premium Filler contains: one pre-filled syringe containing the product, 1mL-graduated syringe, for Infini Premium Filler S , Infini Premium Filler SOFT LIPS, Infini Premium Filler M, Infini Premium Filler L , Infini Premium Filler XL, two 1mL graduated pre-filled syringes containing the product for Infini Premium filler AQUABOOSTER, while it is 10mL-graduated for Infini Premium Filler B BODY. The syringe comes with a Luer-Lock attachment inside a blister and is provided with an adhesive label. A set of two adhesive labels showing the batch number and expiry date, one of which must be applied to the patient's medical file and the other must be given to the patient. These procedures must be strictly performed to ensure product traceability.

Instructions for use to be read carefully before using the device.

Two 30G needles for Infini Premium Filler S, Infini Premium Filler SOFT LIPS and four 30G needles for Infini Premium Filler A AQUABOOSTER, two 27G needles for Infini Premium Filler M and Infini Premium Filler L , two 27G needles for Infini Premium Filler XL.

Storage conditions

Read the expiry date of the product on the adhesive label of the syringe and on the packaging. Store in a dry place, away from direct light and at a temperature between 5°C-25°C.

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